
PressureGuard Easy Air® LR

Microclimate Management with Alternating Pressure & Lateral Rotation

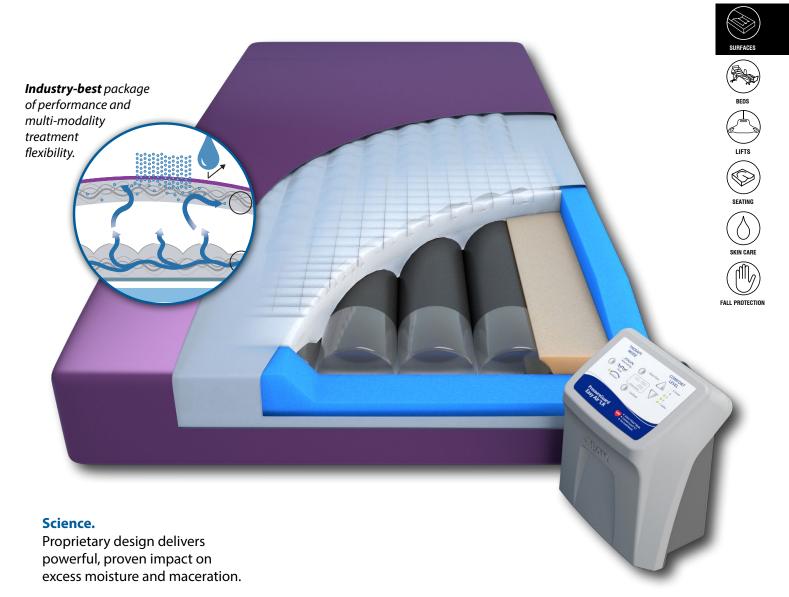

Safety.

Minimizes risk associated with transfers, edge-of-bed/entrapment issues, and power loss.

Simplicity.

Ease of operation and cleaning make it equally user-friendly in facility or at home.




PressureGuard Easy Air® LR

Packed with the design innovations that have made Span a recognized pioneer in high-performance microclimate management and pressure management systems:

Outer Coverlet

- Highly vapor-permeable
- Superior air flow
- Wipes clean in place with standard hospital-grade cleaners
- Zippered, machine-launderable (air dry)

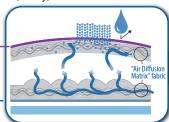


Inner Air Delivery Cover

- Dedicated air supply
- Directs constant stream of air
- Sweeps away moisture vapor before it can re-form as liquid
- Reduces maceration

Proprietary Air Diffusion Matrix™


three-dimensional liner helps ensure uninterrupted air flow in both cover and coverlet

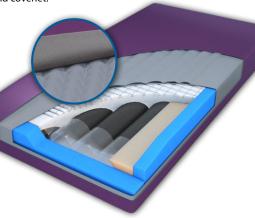

PressureGuard Design: Integrated air system/engineered foam shell with "Safety Edge" bolster system delivers superior edgeof-bed safety, stability and comfort.

Patient indications:

- Patients with excessive skin moisture due to sweating, decreased mobility, inability or refusal to be repositioned.
- Patients with macerated skin due to any of the above.
- Patients with increased skin or body temperature due to infection, sepsis, or other conditions.
- Patients with a combined score \leq 5 on Braden Moisture, Activity and Mobility scales.
- Treatment of Stage 1-4 Pressure Injuries.
- Prevention of Pressure injuries for high-risk patients.
- Some patients with stabilized flaps/grafts (see Span algorithm for parameters).

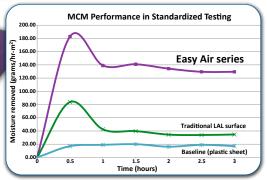

Outstanding microclimate management


performance in controlled, standardized testing [Vol.1: Requirements and Test Methods for Full Body Support Surfaces: American National Standards Institute (ANSI)/RESNA SS-1:2019)].

Specialty model

accommodates recessed deck ("V-deck") frames.



Treatment Flexibility: Select one of three therapy modes:

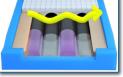


Alternating Pressure gently changes loading across the surface (15-minute cycle).




"Float" mode provides active, powered flotation therapy.



Lateral Rotation, a versatile modality indicated for a variety of needs (15-minute cycle).

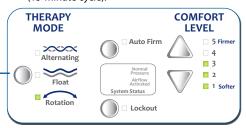

- Does not deflate during power interruption and transport.
- Timed (30-minute) Auto Firm mode for added stability during transfers and ADL.
- Simple comfort adjustment, including ultra-soft "Max Immersion" setting for users under 120 lbs.
- Lockout function eliminates accidental or unintentional changes.
- Normal/Low Pressure indicator.
- Streamlined, lightweight control unit.
- Efficient: Uses less than 15 watts.
- Generates less heat, noise than traditional roll-up surfaces.
- Recessed, shielded connectors for protection from impact, damage.
- Detachable power cord for easy replacement.

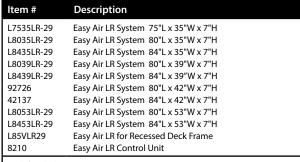

Weight Limit:

500 lbs all models (350 lbs effective limit in Lateral Rotation mode). 39", 42" and 53" widths accommodate wider, non-bariatric beds for users up to 500 lbs. Bariatric (Easy Air XL) models also available for users up to 1000 lbs.

Conforms to NFPA 101 (Life Safety Code); Cal. TB 117; ASTM E1590; and 16 CFR Parts 1632 and 1633.

Complete System Includes Mattress, Control Unit, and Coverlet.

To order coverlet only, add "CLT-" to part number (example: CLT-L8035LR). Mattress Weight approx. 22 lbs. Two-year warranty (all components).

Dimensions: 12" x 9.5" x 7" - Weight: 6 lbs. - Leakage Current: <250micro-amps Voltage: 120 AC - Max Current: 1.0 Amp Frequency: 60 Hertz

Conforms to: ANSI/AAMI ES60601-1:2005 + C1:2009 + A2:2010 + A1:2012 CAN/CSA-C22.2 NO. 60601-1:2014; EN 60601-1:2006 + A1:2013; ANSI/AAMI HA60601-1-11:2015: CAN/CSA-C22.2 NO. 60601-1-11:2015: IEC 6060101-2:2014 (4th ed.): EN 60601-1-2:2014 (4th ed.)



www.spanamerica.com